



# The Springbok brings you confidence and style.

You'll need fewer push strokes through the day with the swept-in frame's rigidity and excellent energy transfer.



Beefy tires and casters are a comfortable ride, even if your day takes you away from polished floors.

The chair is built for a great fit to your needs and for extreme reliability. You'll be amazed at the fast setup, easy maintenance, and durability.



Designed by Participant in California 





## Accessories

Headrest has vertical, forward and backward, and pivot adjustments. Attaches with knobs to the push handles on the back canes. Includes a ledge for the occipital lobe of the user's head.



These lateral trunk supports bolt on to the back canes. Each pad is adjustable for depth, angle, height and width. The pads have a 4 x 5 array of holes on the back. Each pad includes a washable fabric cover.

The activity tray supports a range of activities like fine motor exercises, eating, and education. The design includes two indentions that are great for objects like a cup, a perimeter rim prevents objects from rolling off. The polypropylene material is food safe and super easy to clean.



Bolt on anti tipper. Adjustable and flip up.

Tubular armrests with quick release attachments to the frame.






Footrest Riser. Bracket and foot plate that lifts the footrest surface by 80 mm.



# Springbok Measurements

mm (or as noted)

Seat Width	250	300	350	400	450	500
Seat Depth	250, 300, 350	300, 350, 400	350, 400, 450			
Back Height	300, 350, or 400. The Tall Back Kit: 400, 450, and 500.					
Back Angle	7 angle settings from 78 to 126 degrees at 6-degree intervals.					
Seat Height	470, 490, 510					
Seat Angle from the Ground	7 degrees					
Footrest Height	280 - 380. Footrest Riser Kit: 140 - 300.					
Wheelbase Length	345	395	445			
Overall Width	480	530	580	630	680	730
Overall Length	810	860	910			
Product Weight	11.5 kg	12.7 kg	13.1 kg	14.2 kg	15.7 kg	16.3 kg
Weight Capacity	130 kg					
Center of Gravity Adjustability	Adjustable at 7 settings each 12 mm apart.					
Overall Height	770 mm (Measured with Back Height set at 300 mm, Front seat set at 470 mm)					
Push Handle Height	750 mm (Back height set to 300 mm, Front seat set at 470 mm)					
Rear Wheel Dimensions	600 x 38 mm diameter (24 x 1-3/8 in. 32-540 tire)					
Rear Wheel Camber Angle	4 degrees					
Caster Dimensions	127 mm diameter and 50 mm wide PU tire					
Packaging	One unit per carton; partially assembled; 10 minutes assembly time required.					
Package Size	650 x 700 x 500					
Package Weight	14 kg	15 kg	16 kg	16 kg	18 kg	18 kg
Certifications and Testing	ISO 9001, ISO 13485, CE, Tested ISO 7176 complete, 2X on double drum durability test - 400,000 cycles, ISWP Caster Test - Pass, FWM Wheelchair Test Track - Pass. (see website for all tests and test reports)					

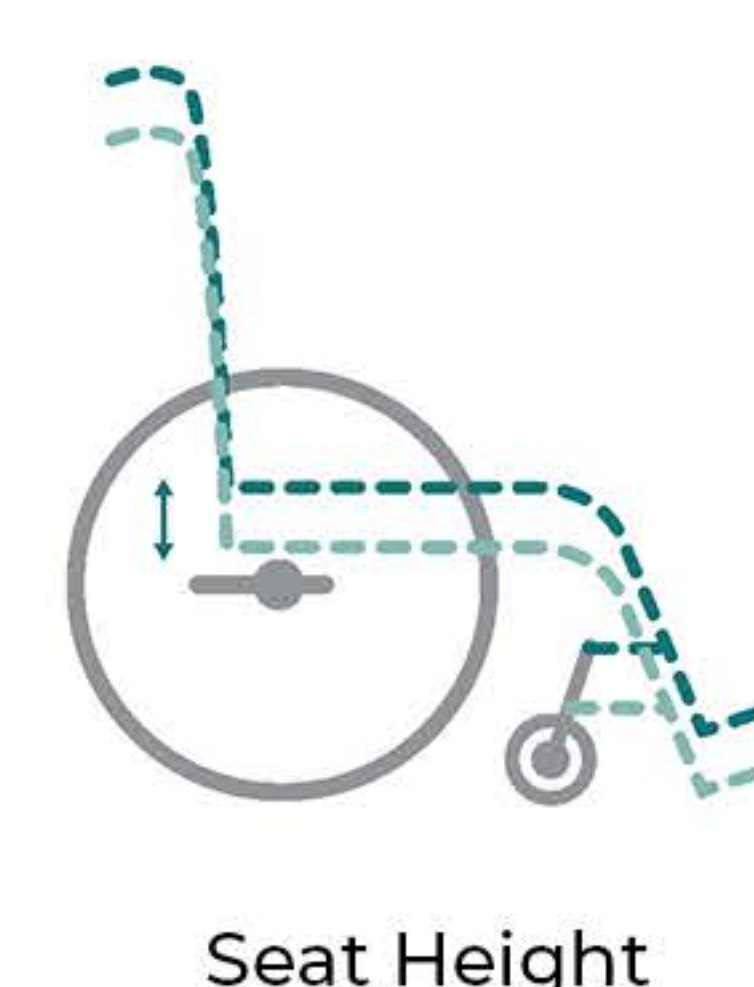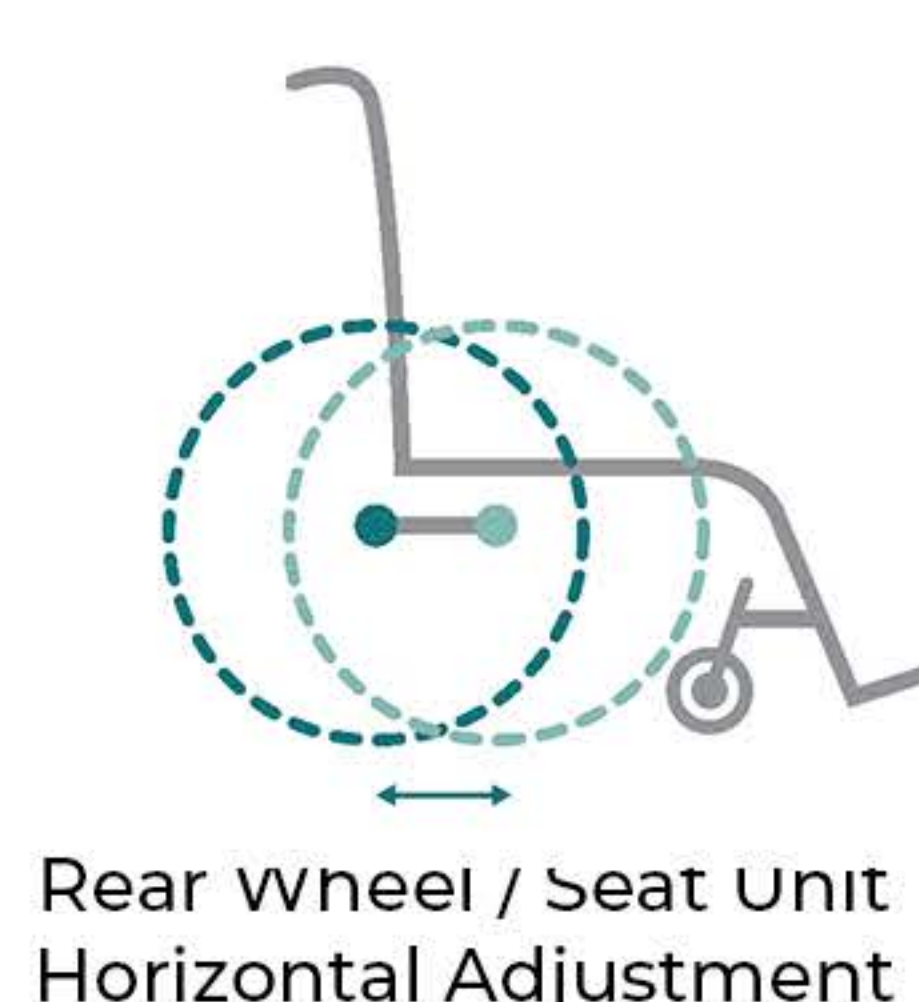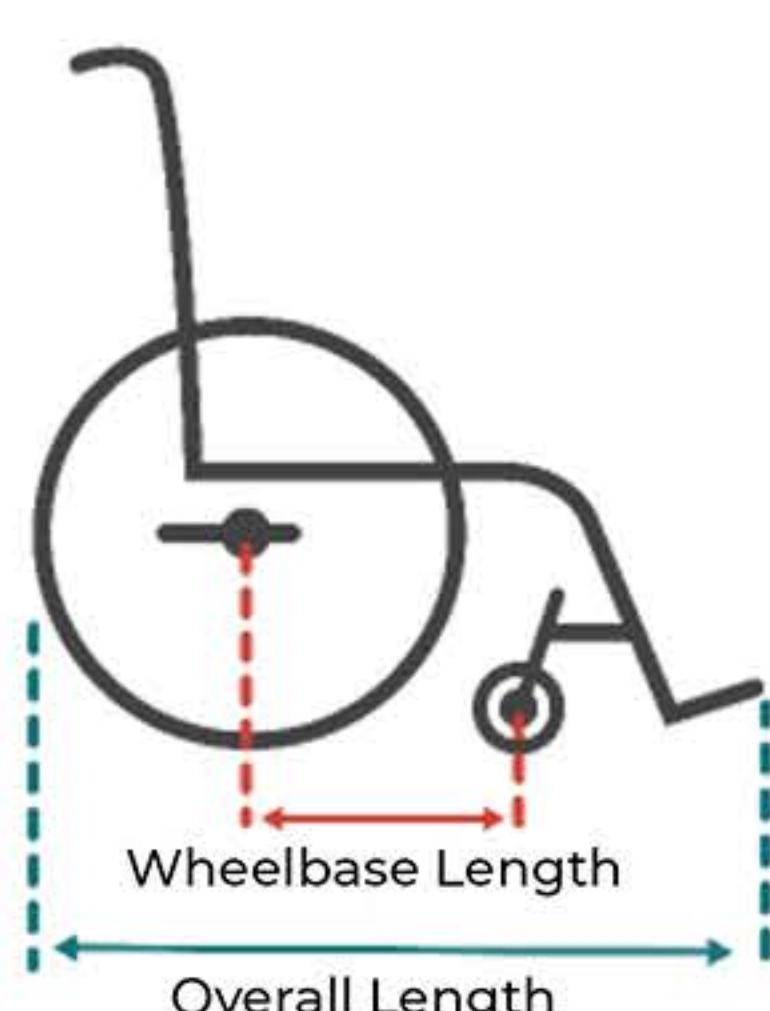

keoke@participant.life | +01 415 218 8278 | Skype: keoke.king

Available August 2021. Pre-orders open June 2021.  
 Field demonstrations are available.



# Fit Kit

## Items included:

- 1 piece hard plastic sheet - 15 x 12 x 1/4" HDPE hard plastic sheet. This can be used to make a seat extension, a mono footplate a wide headrest, to adapt laterals or for other purposes. The plastic is heat gun friendly, is easy to cut with a razor or hack saw, and can be sanded.
- 2 pieces firm foam sheet - 15 x 12 x 1/4" EVA foam. Can be cut and stacked up to make pre-ischial, sacral pad, or other modification.
- 2 tubes of rubber cement - Can be used to permanently glue EVA, HDPE, etc.
- 1 piece of firm foam - 15" x 16" x 2". Polyurethane foam. The density is firmer than the seat cushion. Can be used to make a lift for the seat cushion.
- 1 firm foam wedge, 2" Height x 8" Deep x 15" Wide,
- Hardware - M6 screws, nuts, washers.
- Razor knife.
- 10mm nut driver.

